

RAYCAST® HIGH PRECISION BASE PLATE STORAGE TRAY

INSTRUCTIONS FOR USE

BASE PLATE STORAGE TRAY

Article No.: **32044**



A. GENERAL PRODUCT INFORMATION

The product referred to in this document is a storage system for medical devices, used for patient positioning and immobilisation in radiation therapy.

B. PRODUCT DESCRIPTION

This STORAGE TRAY DEVICE is used to store THE AIO SOLUTION®, IMRT, head, head & neck and pelvic base plates in an upright position.

C. PARTS LIST

The STORAGE TRAY FOR BASE PLATES set Art. no 32044 contains the following parts:

ART. NO.	DESCRIPTION
32044	containing:
32044-1	1x Storage Tray Support Plate
32044-2	2x Storage Tray Side Supports
32044-5	1x Orfit Sticker
32044-6	8x Bolt M6 x 25 mm
32044-7	8x Nut M6
32044-8	16x Washer M6

D. USAGE

Place the Storage Tray in a free area with the Orfit sticker facing the front. Keep the device minimal 1 cm (0,04 inch) from the back wall. Place the base plate into the Storage Tray slot. Do not drop the base plate into the slot as it might damage the base plate.



The Storage Tray can easily be moved using the cut-out area at the back side. Do not try to move the Storage Tray with a base plate inside.

E. STORAGE

Always store the product in a safe place to prevent it from getting damaged.

F. PROPERTIES

Physical Properties

The following physical properties apply to these units:

Dimensions: D 315 mm x W 585 mm x H 600 mm

Weight: 6,37 kg

G. MAINTENANCE, CLEANING AND WASTE MANAGEMENT

This product can be cleaned and disinfected by means of soapy water or an isopropanol based disinfectant, applied with a soft cloth. If unsure about the cleaning fluid, do not use. **Never use aerosol sprays, corrosive cleaning agents, solvents or abrasive detergents.**

Periodic checks of the product should be done to insure the parts are not worn and require repair or replacement. **Do not attempt to make repairs yourself.** Contact your distributor if there are any questions or concerns.

The product can be disposed of with household waste.

H. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Material Safety Data Sheets and regulatory information, please visit our website www.orfit.com.

The instructions were written in accordance with the European Directive 93/42/EEC for Medical Devices. It is prohibited to make alterations to this text without prior approval from ORFIT INDUSTRIES N.V.

RAYCAST® and THE AIO SOLUTION® are registered trademarks of ORFIT INDUSTRIES N.V.

© ORFIT INDUSTRIES N.V.

Ref. No. 31961 – 32044.doc

VERSION 1 - LAST UPDATE: 6/07/2009

REVISION DATE: 06/07/2011



Listed with



www.orfit.com
welcome@orfit.com
radiotherapie.deutschland@orfit.com
sales-us@orfit.com
Vosveld 9a – 2110 Wijmegem – Belgium
Altendorfer Kirchgasse 21 – 99734 Nordhausen
350 Jericho Turnpike, Suite 101 – Jericho – New York 11753 – U.S.A
ORFIT INDUSTRIES N.V.
ORFIT INDUSTRIES N.V. - Vertrieb Deutschland Radiotherapie
ORFIT® INDUSTRIES AMERICA