



HIGH PRECISION BASE PLATES

HIGH PRECISION SPACER FOR HEAD RESTS

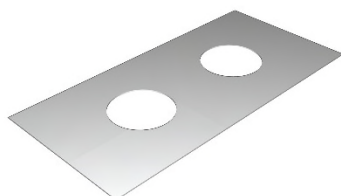

Head only
Article No.: **35702/2** (2 mm thick)

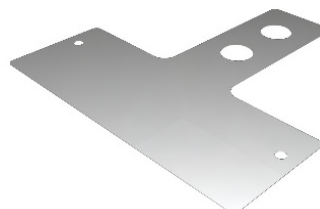

Head & Shoulder
Article No.: **35702/4** (2 mm thick)

A. GENERAL PRODUCT INFORMATION

The products referred to in these instructions are medical devices used for patient positioning and immobilisation in radiation therapy.

This product may only be used in combination with Orfit immobilisation masks products. Orfit prohibits the use of unauthorised products in conjunction with its own products.

B. PRODUCT DESCRIPTION

The SPACERS in transparent plastic are used in combination with the HIGH PRECISION BASE PLATES, HP HEAD SUPPORTS and EFFICAST® PRE-CUTS to form a reproducible patient positioning and immobilisation device. Information on these other parts and instructions on how to make the masks can be found in the respective 'instructions for use' and on www.orfit.com.

The SPACERS can be placed in between the base plates and the head supports while making the thermoplastic mask to reduce the fixation force of the masks on the head of the patient during the treatment fractions.

C. PRECAUTIONS FOR USE

These devices fit around the two knobs on the bottom of the head supports.
Always verify that the devices are positioned correctly on these base plates.

D. STORAGE

Always store the product in a safe place to prevent them from getting damaged. Do not put heavy objects on the devices and prevent hard objects from falling onto it, because these will cause permanent damage.

Store the devices between +10°C (50°F) and 40°C (122°F).

E. PROPERTIES

E.1. Basic Physical Properties

The following physical properties apply to these devices:

Head only

Dimensions:

Length 250 mm x Width 120 mm x Thickness 2 mm

Weight: +/- 50 g

Head & Shoulder

Dimensions:

Length 443 mm x Width 450 mm x Thickness 2 mm

Weight: +/- 200 g

E.2. Mechanical Properties

The spacers are made of PETG clear.

F. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website www.orfit.com.

Note:

It is prohibited to make alterations to this text without prior approval from Orfit Industries.
EFFICAST® is a registered trademark of Orfit Industries.

