


THE AIO SOLUTION® BELLYBOARD & PELVIC SOLUTION BELLYBOARD HAND GRIPS

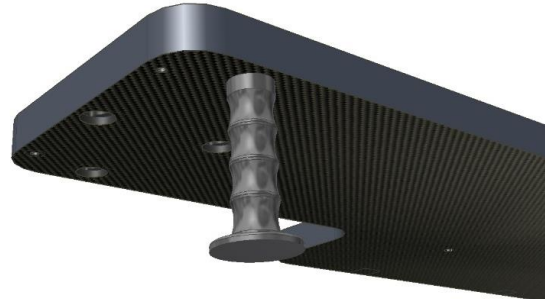

AIO SOLUTION® code: **AC-BG**

Article No. : **32015/8** (packaging : 2 pcs.)

A. GENERAL PRODUCT INFORMATION

The product referred to in these instructions is a medical device used for patient positioning and immobilisation in radiation therapy.

This product may only be used in combination with Orfit immobilisation products. Orfit prohibits the use of unauthorised third-party products in conjunction with its own products.

B. PRODUCT DESCRIPTION

The BELLYBOARD HAND GRIPS are a part of the "AIO SOLUTION® BELLYBOARD PACKAGE". They are used in combination with THE AIO SOLUTION® BASE PLATES, the BELLYBOARD CUSHIONS and the PELVICAST™ PRE-CUTS to form a reproducible patient positioning and immobilisation system. Information on these other parts and instructions on how to make the masks can be found in the respective 'instructions for use' and on www.orfit.com.

C. PRECAUTIONS FOR USE

Always use these BELLYBOARD HAND GRIPS in combination with the other AIO SOLUTION® BELLYBOARD components on one of the AIO SOLUTION Base Plates.

This product consists of two hand grips that are positioned on the cranial part of the AIO Base Plate. Each hand Grip comes with a special fixation screw that can be inserted in the B and D grip pole inserts. The Hand Grip itself can then be attached to the screw from beneath the base plate (to do this the base plate should be positioned in an overhang position on the couch).

Make sure that the AIO base plate is securely fixed to the table top or treatment couch, before positioning the Hand Grips.

Always verify that the device is positioned correctly on the base plate.



D. STORAGE

Always store the product in a safe place to prevent it from getting damaged. Store the product in such a way that it cannot fall or bump against anything. Never place the device under load or pressure.

Store the device between + 10°C (50°F) and 40°C (122°F).

E. PROPERTIES

E.1. Basic Physical Properties

The following physical properties apply to the BELLYBOARD HAND GRIPS:

Dimensions: L 105 mm x ø 55 mm
Weight: 0.25 kg

E.2. Mechanical Properties

The HAND GRIPS are made of POM. This makes sure that they will keep their dimensions overtime and under conditions of frequent use.

F. MAINTENANCE AND WASTE MANAGEMENT

This product can be cleaned and disinfected by means of soapy water or an isopropanol based disinfectant, applied with a soft cloth. If unsure about the cleaning fluid, do not use. **Never use aerosol sprays, corrosive cleaning agents, solvents or abrasive detergents.**

Periodic checks of the product should be done to insure the parts are not worn and require repair or replacement. **Do not**

attempt to make repairs yourself. Contact your distributor if there are any questions or concerns. The product can be disposed of with household waste.

G. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website www.orfit.com.

Note:

The instructions were written in accordance with the European Directive 93/42/EEC for Medical Devices. It is prohibited to make alterations to this text without prior approval from Orfit Industries. THE AIO SOLUTION® is a registered trademark of ORFIT INDUSTRIES N.V. PELVICAST™ is a trademark of ORFIT INDUSTRIES N.V.



ORFIT INDUSTRIES
Vosveld 9A | B-2110 Wijnegem | Belgium
T (+32) (0)3 326 20 26
welcome@orfit.com

www.orfit.com



Ref. No. 31953
VERSION 3
LAST UPDATE: 06/03/2019
REVISION DATE: 06/03/2021