

HIGH PRECISION BASE PLATES

HIGH PRECISION STORAGE STAND FOR BASE PLATES



Article No. : **32044**

A. GENERAL PRODUCT INFORMATION

The product referred to in these instructions is a storage system for medical devices used for patient positioning and immobilisation in radiation therapy.


This product may only be used in combination with Orfit immobilisation products. Orfit prohibits the use of unauthorised third-party products in conjunction with its own products.

B. PRODUCT DESCRIPTION

This STORAGE STAND FOR BASE PLATES is used to store THE AIO SOLUTION®, IMRT, HP PRO®, head, head & neck and pelvic base plates in an upright position.

C. USAGE

Place the Storage Stand in a free area with the Orfit sticker facing the front. Keep the device minimal 1 cm (0,04 inch) from the back wall. Place the base plate into the Storage Stand slot. Do not drop the base plate into the slot as it might damage the base plate.

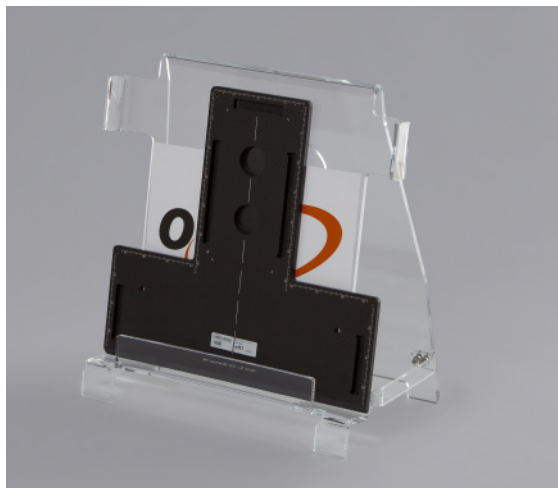

The Storage Stand can easily be moved using the cut-out area at the back side. Do not try to move the Storage Stand with a base plate inside.

D. STORAGE

Always store the product in a safe place to prevent it from getting damaged or falling onto other objects. Store the device between +10°C (50°F) and 40°C (122°F).

E. PROPERTIES

Physical Properties

The following physical properties apply to these units:

Dimensions: D 315 mm x W 585 mm x H 600 mm
Weight: 6,37 kg

F. MAINTENANCE AND WASTE MANAGEMENT

This product can be cleaned and disinfected by means of soapy water or an isopropanol based disinfectant, applied with a soft cloth. If unsure about the cleaning fluid, do not use. **Never use aerosol sprays, corrosive cleaning agents, solvents or abrasive detergents.**

Periodic checks of the product should be done to insure the parts are not worn and require repair or replacement. **Do not attempt to make repairs yourself.** Contact your distributor if there are any questions or concerns.

The product can be disposed of with household waste.

G. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website www.orfit.com.

Note:

It is prohibited to make alterations to this text without prior approval from Orfit Industries. THE AIO SOLUTION® and HP PRO® are registered trademarks of ORFIT INDUSTRIES N.V.



ORFIT INDUSTRIES
Vosveld 9A | B-2110 Wijnegem | Belgium
T (+32) (0)3 326 20 26
welcome@orfit.com

www.orfit.com



Ref. No. 31961
VERSION 5
LAST UPDATE: 21/03/2023
REVISION DATE: 21/03/2025