


## FULL BODY CUSHION FOR CYBERKNIFE COUCH TOP

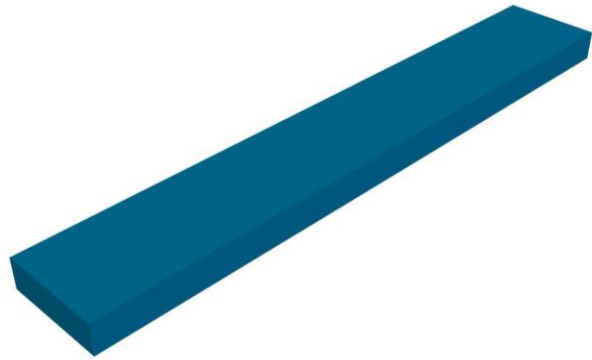

Article No. : **32118**

### A. GENERAL PRODUCT INFORMATION

The products referred to in these instructions are medical devices, used for patient positioning and immobilisation in radiation therapy.

To attain an optimal result, it is recommended to use this product in combination with Orfit immobilisation products.

### B. PRODUCT DESCRIPTION

The cushion is designed to use on a Cyberknife for spinal treatment. The patient is positioned supine on the cushion with the arms along the side of the body. Due to the narrow width of the cushion, the arms will be positioned next to the cushion on the treatment table, instead of on top of the cushion. This allows the use of lateral fields to radiate the spine without the arms being located in the treatment field.



### C. PROPERTIES

The cushion is manufactured from a Polyether foam.

Dimensions: L 2100mm x W 330mm x H 100mm  
Weight: 6.24 kg  
Density: 90 kg/m<sup>3</sup>

### D. STORAGE

Always store the product in a safe place to prevent it from getting damaged. Do not put heavy objects on the cushion and prevent hard objects from falling onto it, because these will cause permanent deformations. Make sure that no sharp objects can come into contact with this cushion. Store the cushions between +10°C (50°F) and 40°C (122°F).

### E. MAINTENANCE AND WASTE MANAGEMENT

These products can be cleaned and disinfected by means of an isopropanol or ethanol based disinfectant, applied with a soft cloth. If unsure about the cleaning fluid, do not use. **Never use aerosol sprays, corrosive cleaning agents, solvents or abrasive detergents. Do not soak the cushion.** Further cleaning instructions can be found in the [Orfit Cleaning Guidelines](#). Contact your distributor if there are any questions or concerns.

### F. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website [www.orfit.com](http://www.orfit.com).

#### Note:

It is prohibited to make alterations to this text without prior approval from Orfit Industries.



**ORFIT INDUSTRIES**  
Vosveld 9A | B-2110 Wijnegem | Belgium  
T (+32) (0)3 326 20 26  
[welcome@orfit.com](mailto:welcome@orfit.com)

[www.orfit.com](http://www.orfit.com)



Ref. No. 50035  
VERSION 5  
LAST UPDATE: 02/04/2021  
REVISION DATE: 02/04/2023