


# RAYCAST® HIGH PRECISION BASE PLATE-TO-COUCH FIXATION DEVICES: 2-PIN BAR FOR 530 MM WIDE NON-INDEXED COUCHES

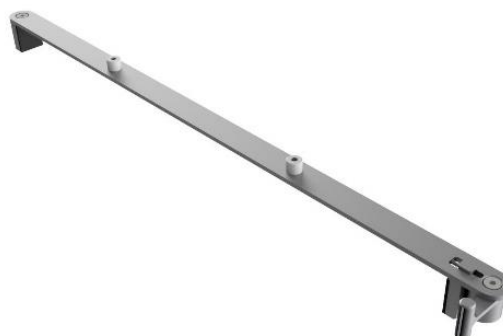

Article No. : **33109**

## A. GENERAL PRODUCT INFORMATION

This product is a fixation device to attach RAYCAST High Precision base plates to non-indexed couch tops with a width of 53 cm. It can also be used to attach base plates to any other couch top which is 53 cm wide although the base plate will not be fixed in longitudinal direction.

## B. PRODUCT DESCRIPTION


This 2-pin bar can be used to index hardware (base plates, shoulder positioning device, leg indexing system, etc.) directly or via an adapter plate to the couch. Information regarding compatible products can be found in the respective 'instructions for use'.

## C. PART LIST

1x	33109	2-pin bar in aluminium
----	-------	------------------------

## D. POSITIONING HARDWARE


To mount the 2-pin bar on the couch, first position the swivel as shown on below picture.

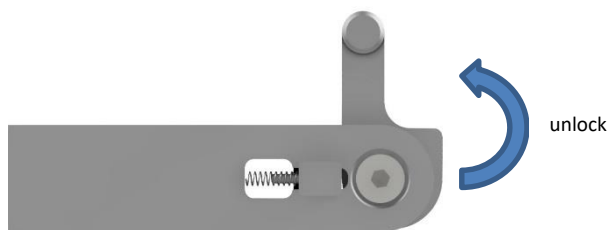

Next position the 2 pin bar on the couch and lock by turning the swivel 180° into locking position.



Orfit hardware is mounted on the 2-pin bar by simply pushing it over the 2 middle indexing pins.

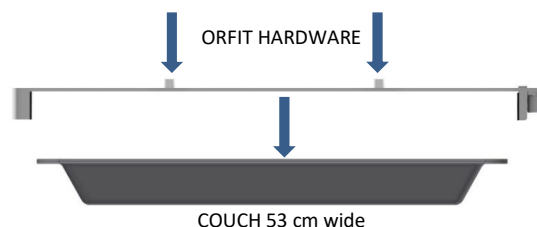

## E. STORAGE

Always store the product in a safe place to prevent it from getting damaged.  
Store the product between +10°C (50°F) and 40°C (122°F).

## F. PROPERTIES

### Physical Properties

Dimensions : L 589 mm x W 51,5 mm x H 44,5 mm  
Weight: 0.16 kg

### Material Properties

Aluminium (blocks/swivel/bar):	2,7 g/cm <sup>3</sup>
Titanium (screws):	4,8 g/cm <sup>3</sup>
Polycarbonate (screws):	1,2 g/cm <sup>3</sup>
Polyoxymethylene (pins):	1,4 g/cm <sup>3</sup>
Rubber (friction blocks):	1,16 g/cm <sup>3</sup>

## G. MAINTENANCE AND WASTE MANAGEMENT

This product can be cleaned and disinfected by means of soapy water or an isopropanol based disinfectant, applied with a soft cloth. If unsure about the cleaning fluid, do not use. **Never use aerosol sprays, corrosive cleaning agents, solvents or abrasive detergents.**

Periodic checks of the product should be done to insure the parts are not worn and require repair or replacement. **Do not attempt to make repairs yourself.** Contact your distributor if there are any questions or concerns.

The product can be disposed of with household waste.

#### H. FIXATION DEVICES COMBINATION PACKAGES

This product is part of following fixation devices combination packages:

- 33120
- 33122
- 33142
- 33125

#### I. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website [www.orfit.com](http://www.orfit.com).

#### Note:

The instructions were written in accordance with the European Directive 93/42/EEC for Medical Devices. It is prohibited to make alterations to this text without prior approval from Orfit Industries. RAYCAST® is a registered trademark of ORFIT INDUSTRIES.



**ORFIT INDUSTRIES**  
Vosveld 9A | B-2110 Wijnegem | Belgium  
T (+32) (0)3 326 20 26  
welcome@orfit.com

[www.orfit.com](http://www.orfit.com)



Ref. No. 50059  
VERSION 5  
LAST UPDATE: 22/03/2018  
REVISION DATE: 22/03/2020