

# RAYCAST® HIGH PRECISION UNILATERAL BREAST HOLDER



Article No. : **18101**

## A. GENERAL PRODUCT INFORMATION

The products referred to in these instructions are medical devices, used for patient positioning and immobilization in radiation therapy.

This product may only be used in combination with Orfit immobilisation products. Orfit prohibits the use of unauthorised third-party products in conjunction with its own products.

## B. PRODUCT DESCRIPTION

The Unilateral Breast Holder is used during breast treatment to pull the contra-lateral breast away from the ipsi-lateral breast. It is used in combination with components of the AIO PRONE BREAST BOARD SOLUTION. Together with these parts they form a reproducible patient positioning and immobilization system. Information on these other parts can be found in the respective 'instructions for use' and on [www.orfit.com](http://www.orfit.com).

The Unilateral Breast Holder is available in different sizes. See also back of this page 'HOW TO FIND THE CORRECT SIZE OF THE UNILATERAL BREAST HOLDER ?'.

| Size | Art. No. |          |
|------|----------|----------|
|      | Left     | Right    |
| S    | 18101/1L | 18101/1R |
| M    | 18101/2L | 18101/2R |
| L    | 18101/3L | 18101/3R |
| XL   | 18101/4L | 18101/4R |
| 2XL  | 18101/5L | 18101/5R |
| 3XL  | 18101/6L | 18101/6R |
| 4XL  | 18101/7L | 18101/7R |

## C. PRECAUTIONS FOR USE

Always make sure that the correct breast holder is used for the correct patient. Mark the patient's details on the breast holder. Clean the breast holder before using it for the next patient.

## D. STORAGE

Always store the product in a safe place to prevent it from getting damaged. Do not put heavy objects on the devices and prevent hard objects from falling onto it, because these will cause permanent damage. Store the devices between +10°C (50°F) and 40°C (122°F).

## E. PROPERTIES

The Breast Holders is made of 100% polyester.

## F. MAINTENANCE AND WASTE MANAGEMENT

The Unilateral Breast Holder can be cleaned by washing it at maximum 30°C (86°F).

Periodic checks of the product should be done to insure the parts are not worn and require repair or replacement. **Do not attempt to make repairs yourself.** Contact your distributor if there are any questions or concerns. The product can be disposed of with household waste.

## G. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website [www.orfit.com](http://www.orfit.com)

# HOW TO FIND THE CORRECT SIZE OF THE UNILATERAL BREAST HOLDER ?

STEP 1: DETERMINE YOUR BRA SIZE (CHEST CIRCUMFERENCE + CUP SIZE)

1. First measure the circumference of the chest right under the breast. This is the first part of your bra size.
2. Now measure the circumference of the breast (cm or inch), by positioning the measuring tape on top of the breast over the nipples. Find your cup size by looking in the column of your chest circumference, for your breast circumference.
3. You will find the unilateral bra size in the right column.

STEP 2: DETERMINE THE SIZE OF THE UNILATERAL BREAST HOLDER BASED ON YOUR BRA SIZE

STEP 3: FIND THE ARTICLE NUMBER OF THE UNILATERAL BREAST HOLDER (see point B. on back of this page)

## EUROPEAN CHEST CIRCUMFERENCE (cm)

| CHEST CIRCUMFERENCE |       |       |       |       |       |       |        |         |         |
|---------------------|-------|-------|-------|-------|-------|-------|--------|---------|---------|
| 63-67               | 68-72 | 73-77 | 78-82 | 83-87 | 88-92 | 93-97 | 98-102 | 103-107 | 108-122 |
| 65                  | 70    | 75    | 80    | 85    | 90    | 95    | 100    | 105     | 110     |

BRA SIZE PART 1

## US CHEST CIRCUMFERENCE (inch)

| CHEST CIRCUMFERENCE |       |       |       |       |       |       |       |       |       |  |  |
|---------------------|-------|-------|-------|-------|-------|-------|-------|-------|-------|--|--|
| 25-26               | 27-28 | 29-30 | 31-32 | 33-34 | 35-36 | 37-38 | 39-40 | 41-42 | 43-44 |  |  |
| 30                  | 32    | 34    | 36    | 38    | 40    | 42    | 44    | 46    | 48    |  |  |

BRA SIZE PART 1

## EUROPEAN CUP SIZE (cm)

|       | 65       | 70       | 75         | 80          | 85          | 90          | 95          | 100         | 105         | 110         |
|-------|----------|----------|------------|-------------|-------------|-------------|-------------|-------------|-------------|-------------|
| CUP A | 77-79 S  | 82-84 S  | 87-89 S    | 92-94 S     | 97-99 S     | 102-104 M   | 107-109 M   | 112-114 L   | 117-119 L   | 122-124 L   |
| CUP B | 79-81 S  | 84-86 S  | 89-91 S    | 94-96 S     | 99-101 M    | 104-106 M   | 109-111 L   | 114-116 L   | 119-121 L   | 124-126 XL  |
| CUP C | 81-83 S  | 86-88 S  | 91-93 S    | 96-98 M     | 101-103 M   | 106-108 L   | 111-113 L   | 116-118 L   | 121-123 XL  | 126-128 XL  |
| CUP D | 83-85 S  | 88-90 S  | 93-95 M    | 98-100 M/L  | 103-105 L   | 108-110 L   | 113-115 L   | 118-120 XL  | 123-125 XL  | 128-130 XL  |
| CUP E | 85-87 S  | 90-92 M  | 95-97 M    | 100-102 L   | 105-107 L   | 110-112 L   | 115-117 XL  | 120-122 XL  | 125-127 XL  | 130-132 2XL |
| CUP F | 87-89 M  | 92-94 L  | 97-99 L    | 102-104 L   | 107-109 XL  | 112-114 XL  | 117-119 XL  | 122-124 2XL | 127-129 2XL | 132-134 2XL |
| CUP G | 89-91 L  | 94-96 L  | 99-101 L   | 104-106 L   | 109-111 XL  | 114-116 XL  | 119-121 XL  | 124-126 2XL | 129-131 2XL | 134-136 2XL |
| CUP H | 91-93 XL | 96-98 XL | 101-103 XL | 106-108 2XL | 111-113 2XL | 116-118 2XL | 121-123 3XL | 126-128 3XL | 131-133 3XL | 136-138 3XL |

## US CUP SIZE (inch)

|       | 30       | 32       | 34       | 36        | 38        | 40        | 42        | 44        | 46        | 48        |
|-------|----------|----------|----------|-----------|-----------|-----------|-----------|-----------|-----------|-----------|
| CUP A | 26-27 S  | 28-29 S  | 30-31 S  | 32-33 S   | 34-35 S   | 36-37 M   | 38-39 M   | 40-41 L   | 42-43 L   | 44-45 L   |
| CUP B | 27-28 S  | 29-30 S  | 31-32 S  | 33-34 S   | 35-36 M   | 37-38 M   | 39-40 L   | 41-42 L   | 43-44 L   | 45-46 XL  |
| CUP C | 28-29 S  | 30-31 S  | 32-33 S  | 34-35 M   | 36-37 M   | 38-39 L   | 40-41 L   | 42-43 L   | 44-45 XL  | 46-47 XL  |
| CUP D | 29-30 S  | 31-32 S  | 33-34 M  | 35-36 M/L | 37-38 L   | 39-40 L   | 41-42 L   | 43-44 XL  | 45-46 XL  | 47-48 XL  |
| CUP E | 30-31 S  | 32-33 M  | 34-35 M  | 36-37 L   | 38-39 L   | 40-41 L   | 42-43 XL  | 44-45 XL  | 46-47 XL  | 48-49 2XL |
| CUP F | 31-32 M  | 33-34 L  | 35-36 L  | 37-38 L   | 39-40 XL  | 41-42 XL  | 43-44 XL  | 45-46 2XL | 47-48 2XL | 49-50 2XL |
| CUP G | 32-33 L  | 34-35 L  | 36-37 L  | 38-39 L   | 40-41 XL  | 42-43 XL  | 44-45 XL  | 46-47 2XL | 48-49 2XL | 50-51 2XL |
| CUP H | 33-34 XL | 35-36 XL | 37-38 XL | 39-40 2XL | 41-42 2XL | 43-44 2XL | 45-46 3XL | 47-48 3XL | 49-50 3XL | 51-52 3XL |

### Note:

It is prohibited to make alterations to this text without prior approval from Orfit Industries.  
RAYCAST® and THE AIO SOLUTION® are registered trademarks of ORFIT Industries.



**ORFIT INDUSTRIES**  
Vosveld 9A | B-2110 Wijnegem | Belgium  
T (+32) (0)3 326 20 26  
welcome@orfit.com

[www.orfit.com](http://www.orfit.com)



Ref. No. 50089

VERSION 4

LAST UPDATE: 02/04/2021

REVISION DATE: 02/04/2023