

**RAYCAST® HIGH PRECISION
POSITIONING AIDS FOR ARMS
AND SHOULDERS
RAYCAST® HIGH PRECISION
ARM STRAP**



Article No. : **32107**

A. GENERAL PRODUCT INFORMATION

The products referred to in this document are medical devices, used for patient positioning and immobilization in radiation therapy.

B. PRODUCT DESCRIPTION

This ARM STRAP can be used to increase patient comfort by supporting the arms in a non-indexed, but comfortable position. It can be used in supine and prone position.



C. PRECAUTIONS FOR USE

The length of the ARM STRAP can be adjusted by means of a Velcro to fit different sizes of patients. Always make sure the Velcro is overlapping at least 50mm to ensure a good fixation. Place the ARM STRAP around the patients wrists to get maximum comfort. Make sure the ARM STRAP isn't putting pressure on sensitive areas on the arms or belly/back of the patient.

D. STORAGE

Always store the product in a safe place to prevent it from getting damaged.

E. PROPERTIES

E.1. Physical properties

The following physical properties apply to these units:

Dimensions: L 680-800 mm x W 50 mm x H 35 mm

Weight: 0.143 kg

E.2. Material properties

Cover: Oil resistant vinyl

Velcro: PA

F. MAINTENANCE AND WASTE MANAGEMENT

This product can be cleaned and disinfected by means of soapy water or an isopropanol based disinfectant, applied with a soft cloth. If unsure about the cleaning fluid, do not use. **Never use aerosol sprays, corrosive cleaning agents, solvents or abrasive detergents.**

Periodic checks of the product should be done to insure the parts are not worn and require repair or replacement. **Do not attempt to make repairs yourself.** Contact your distributor if there are any questions or concerns.

The product can be disposed of with household waste.

G. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website www.orfit.com.

Note:

It is prohibited to make alterations to this text without prior approval from Orfit Industries.



ORFIT INDUSTRIES
Vosveld 9A | B-2110 Wijnegem | Belgium
T (+32) (0)3 326 20 26
welcome@orfit.com

www.orfit.com



Ref. No. 50099
VERSION 2
LAST UPDATE: 02/04/2021
REVISION DATE: 02/04/2023