


RAYCAST® HIGH PRECISION BASE PLATE-TO-COUCH FIXATION DEVICES: 2-PIN BAR FOR 52 CM WIDE NON-INDEXED COUCHES

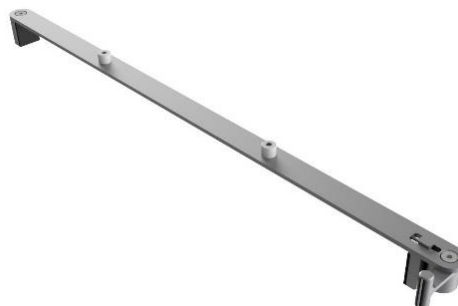

Article No. : 33217

A. GENERAL PRODUCT INFORMATION

This product is a fixation device to attach RAYCAST High Precision base plates to non-indexed couch tops with a width of 52 cm. It can also be used to attach base plates to any other couch which is 52 cm wide although the base plate will not be fixed in longitudinal direction.

B. PRODUCT DESCRIPTION


This 2-pin bar can be used to index hardware (base plates, shoulder positioning device, leg indexing system, etc.) directly or via an adapter plate to the couch. Information regarding compatible products can be found in the respective 'instructions for use' and on www.orfit.com.

C. PART LIST

1x	33217	2-pin bar in aluminium
----	-------	------------------------


D. POSITIONING HARDWARE

To mount the 2-pin bar on the couch, first position the swivel as shown on below picture.

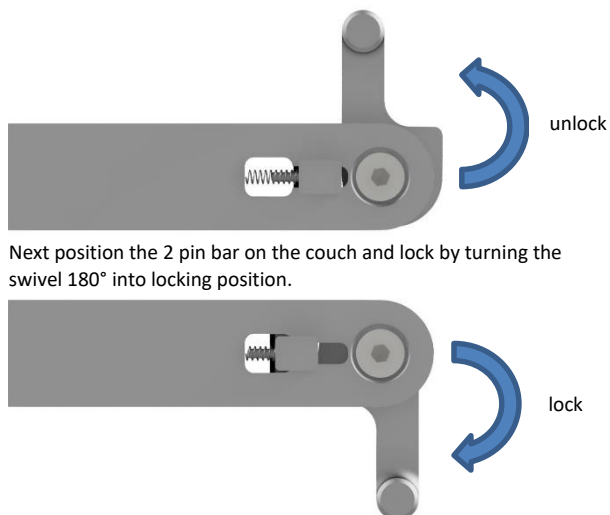

Next position the 2 pin bar on the couch and lock by turning the swivel 180° into locking position.

Orfit hardware is mounted on the 2-pin bar by simply pushing it over the 2 middle indexing pins.

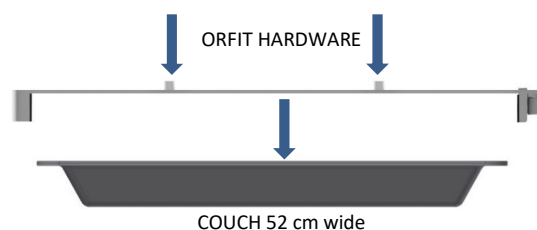

E. STORAGE

Always store the product in a safe place to prevent it from getting damaged.
Store the product between +10°C (50°F) and 40°C (122°F).

F. PROPERTIES

Physical Properties

Dimensions : L 579 mm x W 51,5 mm x H 44,5 mm
Weight: 0.24 kg

Material Properties

Aluminium (blocks/swivel/bar):	2,7 g/cm ³
Titanium (screws):	4,8 g/cm ³
Polycarbonate (screws):	1,2 g/cm ³
Polyoxymethylene (pins):	1,4 g/cm ³
Rubber (friction blocks):	1,16 g/cm ³

G. MAINTENANCE AND WASTE MANAGEMENT

This product can be cleaned and disinfected by means of soapy water or an isopropanol based disinfectant, applied with a soft cloth. If unsure about the cleaning fluid, do not use. **Never use aerosol sprays, corrosive cleaning agents, solvents or abrasive detergents.**

Periodic checks of the product should be done to insure the parts are not worn and require repair or replacement. **Do not attempt to make repairs yourself.** Contact your distributor if there are any questions or concerns.

The product can be disposed of with household waste.

H. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website www.orfit.com.

Note:

The instructions were written in accordance with the European Directive 93/42/EEC for Medical Devices. It is prohibited to make alterations to this text without prior approval from Orfit Industries.
RAYCAST® is a registered trademark of ORFIT Industries.

