

HP PRO[®] SOLUTION HEAD SUPPORT INDEXING PLATE



Article No. : **25000/58/20** (package of 20 pieces)

A. GENERAL PRODUCT INFORMATION

The product referred to in these instructions is a medical device used for patient positioning and immobilisation in radiation therapy.

This product can be used during both the simulation and the treatment stage, including MRI simulation.

This product may only be used in combination with Orfit immobilisation products. Orfit prohibits the use of unauthorised third-party products in conjunction with its own products.

B. PRODUCT DESCRIPTION

The Head Support Indexing Plate is an accessory of the ORFIT HP PRO[®] SOLUTION. It is used in combination with the HP PRO[®] base plate, special 3 and 5 point masks, a cranial stop and a personalised head rest to form a reproducible patient positioning and immobilisation device. Information on these other parts and instructions on how to make the masks can be found in the respective 'instructions for use' and on www.orfit.com.

C. USAGE

The Head Support Indexing Plate is designed to attach a personalised head support, such as a Moldcare cushion, to the base plate. The indexing plate has two pins at the bottom that drop into two holes in the base plate. There are strips on the top surface that can be removed to expose the glue onto which the Moldcare cushion can be attached. The indexing plate can be used with or without shim.

1. Attach the indexing plate to the base plate



2. Remove the strips to expose the glue.



3. Attach the cushion to the indexing plate.



4. Position the patient on the Moldcare cushion and mould it to create enough support for the head and neck. Make sure to push the Moldcare cushion towards the indexing plate at the same time to prevent deformation of the indexing plate. Deformation might cause that the indexing plate pops out of the holes in the HP PRO[®] base plate.

D. PHYSICAL PROPERTIES

25000/58

Dimensions	: L 215 mm x W 116 mm x H 18 mm (curved sheet +/-1,5 mm thick)
Weight	: 0,025 kg (with tape)
Material	: ABS

This product is made entirely of electrically non-conductive, non-metallic and non-magnetic materials and is MRI safe.'



E. STORAGE

Always store the product in a safe place to prevent it from getting damaged. Store it in a dry place at a temperature of min. 10°C (50°F) and max. 30°C (86°F)

F. MAINTENANCE AND WASTE MANAGEMENT

These products can be cleaned and disinfected by means of an isopropanol based disinfectant, applied with a soft cloth. If unsure about the cleaning fluid, do not use. **Never use aerosol sprays, corrosive cleaning agents, solvents or abrasive detergents. Do not soak the products.**

Further cleaning instructions can be found in the [Orfit Cleaning Guidelines](#).

The product can be disposed of with household waste.

G. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website www.orfit.com.

Note:

It is prohibited to make alterations to this text without prior approval from Orfit Industries.
HP PRO® is a registered trademark of ORFIT Industries.



ORFIT INDUSTRIES

Vosveld 9A | B-2110 Wijnegem | Belgium
T (+32) (0)3 326 20 26
welcome@orfit.com

www.orfit.com



Ref. No. 50248

VERSION 1

LAST UPDATE: 26/04/2022

REVISION DATE: 26/04/2024